

# Medtrica Solutions, Ltd

4001 Inglewood Ave; 101-128

Redondo Beach, CA 90278

P: 877-703-1079/F: 310-263-9098

[www.medtrica.com](http://www.medtrica.com)



EWC804100

*Tubular Sponge: w/ Enzymatic Detergent CONCENTRATE, Individually Sealed*

## **Directions for Use:**

1. *Open package using the tear notches and remove the sponge*
2. *Place sponge in container of clean water (up to 500 ml of water)*
3. *Squeeze the sponge 5-7 times in the water container to activate the Enzymatic Detergent*
4. *Squeeze and remove the sponge from the container, then use the sponge to wipe down the exterior of the endoscope*
5. *Place the distal end of the instrument in the container of freshly made Enzymatic Detergent solution and cycle the fluid through the instrument*
6. *Sponge may be taken to the sterilization department for further pre-cleaning of the instrument.*
7. *Discard the sponge and used enzymatic detergent solution using approved methods for materials that have been in contact with possible infectious waste*

## **Product Description:**

*Activate with a minimum of 50 ml and up to 500 ml of water - basin or stand-up pouch not included*

- *Enzymatic Detergent concentrate contains 3 enzymes - Protease, Amylase and Lipase, PLUS Surfactant Detergents to break down proteinaceous bioburden during the preclean of surgical instruments*
- *Bulk Packaged - 100 ct / 1 case - (1 bag of 100)*
- *For precleaning of instruments after a procedure. Low foaming formula*
- *Tube Sponge size: 4.25"L x 2.25"D w/ .25" center hole (105mm x 57mm w/6mm center hole)*
- *Green Friendly, High Density, 100% Medical Grade foam, No Flaking or Scratching*
- *Biodegradable, non-toxic, odorless and non-irritating to the eyes*
- *Notched, easy tear open packaging*