

Medtrica Solutions, Ltd

4001 Inglewood Ave; 101-128

Redondo Beach, CA 90278

P: 877-703-1079/F: 310-263-9098

www.medtrica.com



ESW904100

Tubular Sponge: w/ 30 ml of pre-mixed Enzymatic Detergent SOLUTION, Individually Sealed

Directions for Use:

1. *Open package using the tear notches and remove the sponge*
2. *Remove sponge and place it onto the insertion tube so that the insertion tube is "inside" the sponge.*
3. *Use the sponge to wipe down the exterior of the endoscope*
4. *Tubular sponge may be used as bumper guard around the distal / lens end of the endoscope during transport to the sterilization department for further pre-cleaning of the instrument.*
5. *Discard the sponge using approved methods for materials that have been in contact with possible infectious waste.*

Product Details:

- *For precleaning of instruments after a procedure. Low foaming formula*
- *Enzymatic Detergent concentrate contains 3 enzymes - Protease, Amylase and Lipase, PLUS Surfactant Detergents to break down proteinaceous bioburden during the preclean of surgical instruments*
- *The water that is mixed with enzymatic detergent concentrate goes through a multi-stage cleaning process in the following order:
1) Particulate Filtration; 2) Charcoal Filtration; 3) Semi Permeable Membrane (reverse osmosis); 4) UV Light*
- *Tube Sponge size: 4.25"L x 2.25"D w/ 0.25" center hole (105mm x 57mm w/ 6mm center hole)*
- *Green Friendly, High Density, 100% Medical Grade foam, No Flaking or Scratching*
- *Biodegradable, non-toxic, odorless and non-irritating to the eyes*
- *Notched, easy tear open packaging*
- *Bulk Packaged - 100 ct / 1 case - (1 bag of 100) or Dispenser Box packaged - 4 boxes of 25 ea; 100ct / 1 case*