

Medtrica Solutions Ltd
4001 Inglewood Ave; 101-128
Redondo Beach, CA 90278
P: 877-703-1079/F: 310-263-9098
www.medtrica.com



Tubular DuraSponge® Usage Guidelines – ‘Bumper Guard’

1. *Remove Tubular DuraSponge from box*
2. *Place Tubular DuraSponge on but not over the distal end on the insertion tube of a fully reprocessed dry endoscope – **allow for air-flow around the lens***
3. *Remove Tubular DuraSponge from the endoscope after transporting the scope to the procedure area and just before prepping the scope for the procedure*
4. *Immediately after the procedure soak the Tubular DuraSponge in either an enzymatic detergent solution or water – (follow scope manufacturers recommendations re the liquid) and wipe down the entire insertion tube to remove all visible bioburden*
5. *Place Tubular DuraSponge on but not over the distal end on the insertion tube before transporting the endoscope to the reprocessing area*
6. *In the reprocessing area rinse and use the Tubular DuraSponge for the final precleaning before sterilizing the endoscope*
7. *Do not save or reuse – use the same sponge throughout the entire process then dispose following standard infectious waste protocol*